

Spare Parts

MODEL	PRODUCTS	ORDER CODE	MODEL	PRODUCTS	ORDER CODE
PM-S Handpiece (Short Type)		T706-010	PM-BA For Bien-Air® Unifix® Coupling		T741-020
PM-L Handpiece (Long Type)		T705-010	PM-SR For Sirona® Quick Coupling		T740-020
PM-PTL For NSK FlexiQuik Coupling (PTL-CL)		T705-020	Powder Case Cover	 1. Blue 2. Green 3. Pink	1. T705-133 2. T705-233 3. T705-333
PM-QD For NSK FlexiQuik Coupling (QD-J)		T729-020	Cleaning Powder (100 packages per box)		Y900-052
PM-M4 For Midwest 4 hole joint		T719-020	PTL O-ring Set (Pack of 5)		Y900-580
PM-B2 For Borden 2 / 3 hole joint		T718-020	PM O-ring Set (Pack of 3)		Y900-054
PM-KV For KaVo® Multiflex® Coupling		T739-020	Nozzle		T705A013
PM-WH For W&H® Roto Quick® Coupling		T745-020			



PROPHY-MATE FROM NSK
TOOTH CLEANING HANDPIECES

clean & bright - quickly



Propphy-Mate complete System

COUPLING TYPE	NSK	KaVo®	Sirona®	W&H®	Bien-Air®	M4	B2				
S Short Nozzle	MODEL	PM-PTL-S	PM-QD-S	PM-KV-S	PM-SR-S	PM-WH-S	PM-BA-S	PM-M4-S	PM-B2-S		
	ORDER CODE	Y135-011	Y135-012	Y135-015	Y135-018	Y135-016	Y135-017	Y135-013	Y135-014		
L Long Nozzle	MODEL	PM-PTL-L	PM-QD-L	PM-KV-L	PM-SR-L	PM-WH-L	PM-BA-L	PM-M4-L	PM-B2-L		
	ORDER CODE	Y135-019	Y135-020	Y135-023	Y135-026	Y135-024	Y135-025	Y135-021	Y135-022		
Connect Direct To these Couplings										Connect direct to the handpiece tubing MIDWEST	Connect direct to the handpiece tubing BORDEN
	NSK ORDER CODE	NSK FM-CL-M4 P434-051	NSK FM-CL-B2/B3 P432-051	NSK QD-J M4 T334-051	NSK QD-J B2/B3 T332-051	KaVo® MULTiflex® LUX	Sirona® Quick Coupling	W&H® Roto Quick®	Bien-Air® Unifix®		

- Complete System Includes
- Propphy-Mate Handpiece with short or long nozzle
 - Cleaning brush, wire and file
 - 10 pcs, x 12 g sachets of cleaning powder
 - Replacement nozzle head
 - 3 covers (Blue, Green, Pink) for powder chamber
 - Wrench for Replacing Nozzle

*KaVo® and MULTiflex® are registered trademarks of Kaltentbach & Voigt GmbH & Co., Germany.
Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany.
W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.
Bien-Air® and Unifix® are registered trademarks of Bien-Air Dental S.A., Switzerland.

www.nsk-inc.com			
NAKANISHI INC 700 Shimohinata Kanuma, Tochigi 322-8666, Japan TEL:+81 (0)289-64-3380 FAX:+81 (0)289-62-5636	NSK TOKYO OFFICE 6F MY Building Ueno 1-13-3, Taito-ku Tokyo 110-0005, Japan TEL:+81 (0)3-3835-2892 FAX:+81 (0)3-3835-2856	NSK EUROPE GmbH Westerbachstraße 58 Ueno 1-13-3, Frankfurt, Germany TEL:+49 (0)69 74 22 99 0 FAX:+49 (0)69 74 22 99 29	NSK FRANCE SAS 19 avenue de Villiers D-60489, Frankfurt, France TEL:+33 01 56 79 59 80 FAX:+33 01 56 79 59 81
		NSK MIDDLE EAST Dubai Airport Free Zone, Room 2W-310, 3F West Wing, Dubai, UAE TEL:+971-4-299-6644 FAX:+971-4-299-6664	

Specifications are subject to change without notice.

REF: PR-D133E Ver.6 06.04.03.Y

Powerful Partners®



Prophy-Mate works on the same FlexiQuik couplings as NSK high speed handpieces. It is a lightweight, compact and easy to maintain handpiece for tooth cleaning and polishing. The unique twin jet nozzle system offers free flow of the polishing powder direct to the tooth surface.



Designed for Application to Both Anterior and Posterior Teeth



The carefully designed length, angle, and curve of the nozzle allows accurate direction of the powder jet.

Ergo Comfort Design



The functional handpiece has been designed to fit comfortably into large and small hands. The rounded handpiece fits the hand securely, and the knurled grip allows firm and accurate tip control. Long and short handpieces are available.

Lemon Flavored Cleaning Powder



The jet polishing powder has a refreshing lemon scent. The correct quantity (12 g) of powder for the instrument powder case packed in individual sealed packets. Each packet offers approximately 3 minutes of polishing.

Three Colors for Powder Chamber




Three translucent covers (blue, green and pink) are included as accessories. They are easy to replace so that you can use them as you like.


*Case covers are not autoclavable. Clean with alcohol.




Prophy-Mate S Short Handpiece

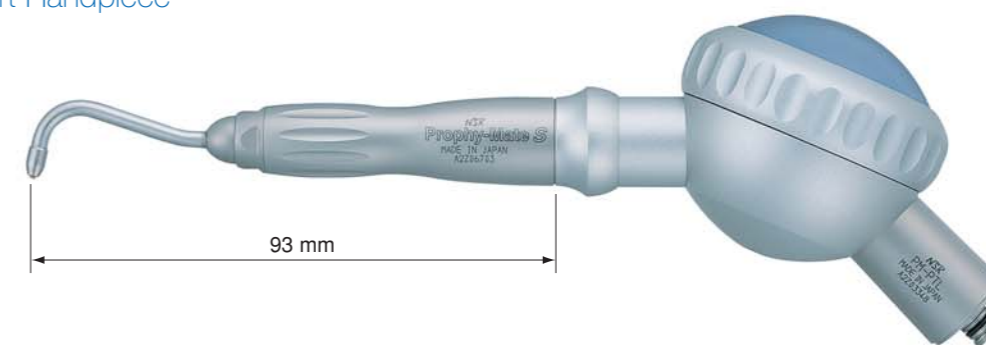

Prophy-Mate L Long Handpiece

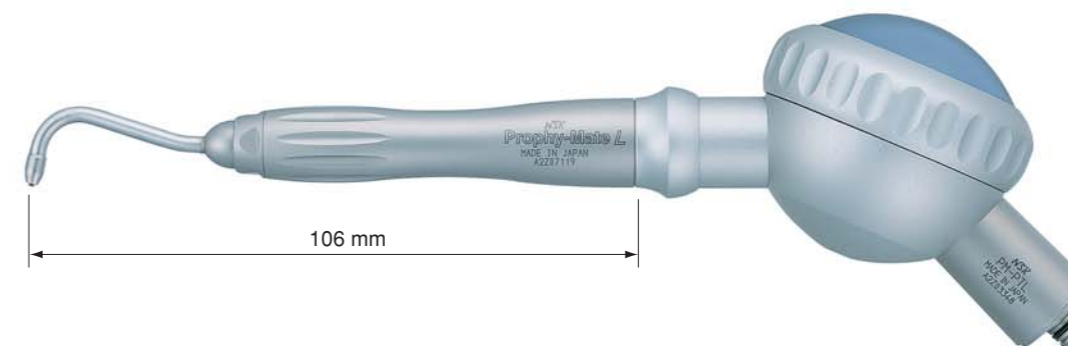

An example of the application of Prophy-Mate

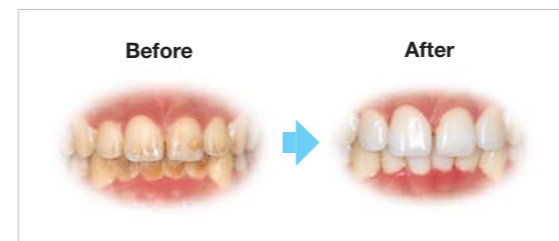

Detachable Nozzle




The nozzle head is made removable for easy cleaning. One nozzle head is included in a pack as an accessory part.

Simple and Quick Connection. Easy Maintenance



Simply pull back the quick release ring to release the Prophy-Mate from the coupling. Maintenance can be done easily with the cleaning brush and wire supplied. Prophy-Mate can be repeatedly autoclaved at up to 135°C after removing the powder case cover.

Lightweight and Flexible Design

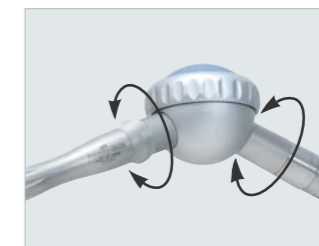

Prophy-Mate turns 360 degrees. The handpiece connection is specially designed to rotate freely even under strong air pressure. Prophy-Mate is optimally angled to reduce hand and wrist fatigue.

NSK Unique Twin Jet Nozzles



Twin jet nozzles simultaneously direct air into the powder case, constantly pushing the powder toward the mid suction nozzle. The curved powder case reduces powder residue. This system provides a stable and precise jet for an excellent polishing effect.

Connects All Brand Quick Couplings



Prophy-Mate is quickly and simply attached to NSK FlexiQuik couplings. Also available for Midwest 4 hole and Borden 2 / 3 hole connection direct to the hose, plus KaVo® Multiflex®, W&H® Roto Quick®, Bien-Air® Unifix® and Sirona® Quick Couplings.